



	Release Note for PEDIARITY 1.2	
	GD00195 vA	PEDIARITY™

# PEDIARITY™ v1.2 Release Note


Version	Date	Change description
0	10-DEC-2024	Initial version
1	11-DEC-2024	Bug monitoring-data#17 confirmed to be solved Bug gabi-hcp-web#781 still present and added
2	12-DEC-2024	Bug monitoring-data#18 detected and added
3	18-DEC-2024	Bug gabi-hcp#783, gabi-hcp#785 and patient-management#27 detected and added
A	06-JAN-2025	Target version based on v3

Role	Name	Date	Signature
<b>Author</b>	Olivier Staquet	08-Jan-2025	DocuSigned by:  FEB5A30AAB3B4CC...
<b>Reviewer</b>	Nathalie Bosschaerts	09-Jan-2025	DocuSigned by:  7F39B238CAED4CC...
<b>Approver</b>	Edouard Carton	08-Jan-2025	DocuSigned by:  C9C131A1D8FE438...

	Release Note for PEDIARITY 1.2	
	GD00195 vA	PEDIARITY™

# 1. Table of contents

<b>1. Table of contents</b>	<b>2</b>
<b>2. Definitions &amp; abbreviations</b>	<b>3</b>
<b>3. Related documents</b>	<b>3</b>
<b>4. Reference documents &amp; norms</b>	<b>3</b>
<b>5. Introduction</b>	<b>4</b>
<b>6. Version of subsystems</b>	<b>4</b>
<b>7. Change log</b>	<b>4</b>
7.1. New features or updated features	4
7.2. Bug fixes	5
7.3. Technical changes	6
<b>8. Known issues</b>	<b>6</b>
8.1. Safety issues	6
8.2. Blocker issues	6
8.3. Minor issues	6
8.4. Cosmetic issues	7

	Release Note for PEDIARITY 1.2	
	GD00195 vA	PEDIARITY™

## 2. Definitions & abbreviations


ACCS	Accelerometers
DPO	Data Protection Officer
HCP	Healthcare professional
IFU	Instructions for use
PR	Pulse rate
RR	Respiratory rate
SpO2	Peripheral oxygen saturation

## 3. Related documents

GD00001	PEDIARITY User Needs
GD00008	PEDIARITY Design requirements
GD00021	PEDIARITY Caregiver IFU
GD00022	PEDIARITY HCP IFU

## 4. Reference documents & norms

IEC 62304:2006	Medical device software – Software life cycle processes
----------------	---

	Release Note for PEDIARITY 1.2	
	GD00195 vA	PEDIARITY™

## 5. Introduction

This document is the release note of PEDIARITY™ v1.2. It covers:

- the versions of the subsystems included in the version
- the change log compared to the previous version (v1.1)
- the known issues in the software.

This document is communicated to the end user of the system.

## 6. Version of subsystems


Subsystem	Component	Version
Gabi Band	Hardware	GSC2 rev. B
	Firmware	1.6.10
	Including Signal processing algorithm	1.4.9
Gabi Monitor App	Application	1.13.6+186
Gabi Analytics	Analytics	0.12.0
	Privacy	0.12.0
Gabi Cloud	Patient Management	1.0.13
	Mobile Connector	1.0.10
	Monitoring Data	1.0.9
	Stack	1.0.5

## 7. Change log

### 7.1. New features or updated features


Patient's dashboard	<ul style="list-style-type: none"> <li>• Improve the speed of display</li> <li>• Display the name of the patients "last name, first name"</li> </ul>
---------------------	--

The detailed features are defined in GD00001 (User needs) and GD00008 (Requirements).

	Release Note for PEDIARITY 1.2	
	GD00195 vA	PEDIARITY™

## 7.2. Bug fixes

Gitlab ID	Description
<a href="#">gabi-hcp-web #394</a>	When HCP updates the language of the user, the mobile app keeps the language selected by the user during the onboarding
<a href="#">gabi-hcp-web #715</a>	In the daily biometrics view, when zooming starts in the middle of an event, the event is not displayed in the zoom windows.
<a href="#">gabi-hcp-web #684</a>	The sorting of the patients' list is case sensitive on the last name of the patient
<a href="#">gabi-hcp-web #386</a>	In the privacy screen, the tab name overlap with the ">" on low resolution screen
<a href="#">gabi-hcp-web #718</a>	There is a link "See full change log history" at the end of the change log which is not needed and inactive
<a href="#">gabi-hcp-web #707</a>	When sorting the patients' list, the GUI doesn't differentiate "no data" and "0 events". It leads to mixed sorting for those data.
<a href="#">gabi-hcp-web #704</a>	Broken image of sitting fox when requesting a date in daily biometrics for the demo patient and for which there is no data.
<a href="#">gabi-hcp-web #703</a>	Data is not available for the patient demo when selecting the date before 01-JAN-2021.
<a href="#">gabi-hcp-web #688</a>	When there is no data in the last column of the patients' list, the hint popup appears outside of the screen (cropped)
<a href="#">gabi-hcp-web #674</a>	On low resolution, some parts are not fully displayed in the title bar (HCP name, Analytics version and logout menu).
<a href="#">gabi-hcp-web #650</a>	On last available night health report, the display of the sorting of events has 3 statuses displayed (1 arrow up, 1 arrow down, 2 arrows up & down). The sorting cycle should alternate between up & down.
<a href="#">gabi-hcp-web #645</a>	The logo of Gabi displayed in the "Too small screen message" is the old one.
<a href="#">gabi-hcp-web #593</a>	After moving the guided tour window and resizing the browser window, the guided tour window may be outside of the visible window.
<a href="#">gabi-hcp-web #572</a>	When expanding the health condition and the patient info at the same time, the patient's info superpose the health condition.
<a href="#">gabi-hcp-web #711</a>	The first time indicated in the preview of an event is shifted.

	Release Note for PEDIARITY 1.2	
	GD00195 vA	PEDIARITY™

### 7.3. Technical changes

Architecture	The Gabi Cloud architecture has been split into different modules to prepare the future evolution of the platform.
Dependency updates	The dependencies have been updated due to vulnerabilities found in some of the dependencies

## 8. Known issues

The known issues are classified into 4 categories:

- Safety issues: All known issues that have an impact on the safety of the patient.
- Blocker issues: Known issues for which there is no known workaround.
- Minor: Other known issues that have no impact on the safety or the performance.
- Cosmetic: Issues that don't impact the usage of the system (e.g. typos, etc). Cosmetic issues have no impact on safety or performance.

The issues are planned for resolution in a future release.

### 8.1. Safety issues


None

### 8.2. Blocker issues

None

### 8.3. Minor issues


Gitlab ID	Description	Workaround
<a href="#">gabi-hcp-web #253</a>	The cursor of the scrollbar is misaligned on Safari.	If using Safari, the cursor allowing you to navigate through the recording may respond slowly. You can use another browser to ease your navigation.
<a href="#">gabi-hcp-web #664</a>	On unstable connectivity, the error 502 could appear (Bad Gateway) which could lead to missing information in the GUI.	When the error appears, reload the web page.
<a href="#">gabi-hcp-web #641</a>	During the creation of the patient, the user may be logged out for inactivity and receive an error	When the error appears, reload the web page.

	Release Note for PEDIARITY 1.2	
	GD00195 vA	PEDIARITY™

	message.	
<a href="#">gabi-hcp-web#781</a>	During patient creation, the back button doesn't trigger the warning for form exit	Ensure to perform the creation of the patient completely to let the user perform the onboarding.
<a href="#">pediarity#69</a>	The biometric is tagged as not reliable in case of flapping. The implementation is considering 28 seconds included instead of 30 seconds not included to define the duration for a stable signal.	None. This is a small update in the definition of the flapping.
<a href="#">gabi-firmware#151</a>	Raw data recovery may be stuck when multiple disconnections during the recovery.	Instruct the user to keep the band close enough to the tablet during a recording.
<a href="#">monitoring-data#18</a>	The event per hour chart in the health report doesn't display if there is less than 1% of events per hour (36 seconds)	The events' rates are correctly displayed in the other screens. Therefore, cross-check with the other values if the chart is empty and the table of events contains entries.
<a href="#">cna-app#502</a>	Sometimes, forced firmware update doesn't work at the first attempt	Shutdown the tablet, unplug the band, start the tablet, plug the band on power and retry the forced update
<a href="#">cna-app#533</a>	If the band is disconnected, the "I need help" message doesn't mention that the band should not be plugged while worn by the child.	The band cannot be worn while plugged on a power plug.
<a href="#">cna-app#534</a>	Special band configuration is not applied on app launch for research studies	Reset the app and redo the onboarding while ensuring that the Internet is available.

## 8.4. Cosmetic issues

Gitlab ID	Description	Workaround
<a href="#">gabi-hcp-web#656</a>	The label "12h" is not displayed in the last available night health report in events' chart while it is displayed properly in the aggregated chart.	None
<a href="#">gabi-hcp-web#783</a>	When there is no reliable data at all for a day, the hover mentions "no recording for this night".	None
<a href="#">gabi-hcp-web#785</a>	When the HCP self-registers, the button with the list of new features stays displayed even after display.	None
<a href="#">pediarity#59</a>	During the onboarding on the mobile app, the first	None

	Release Note for PEDIARITY 1.2	
	GD00195 vA	PEDIARITY™

	name and the band S/N are displayed when the link with the patient is done; not when the link with the band is done.	
<a href="#">cna-app#351</a>	The reminder to start the recording briefly appears when the data are already uploaded to the cloud.	None
<a href="#">patient-management#27</a>	When the caregiver deletes his personal data, the birth date of the child mentioned in the email to the HCP is formatted YYYY-MM-DD instead of DD-MMM-YYYY.	None